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## FAB PMMA

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### 2. GENERAL

**FAB PMMA** milling blanks for making splints and long-lasting temporary crowns and bridges and can be used in all CAD/CAM system. To process use only carbide burs. Please note that the improper use of milling tools, polishing brushes, steam jets and water baths can cause the material to overheat. This can result in damage to the material.

### 3. INDICATION

**FAB PMMA** is a device made from PMMA for the fabrication of splints and temporary crowns and bridges and is intended for use in the oral cavity while awaiting a permanent restoration. Restorations are designed and manufactured by a dental technician using CAD/CAM technology.

### 4. CONTRAINDICATION

Do not use this product in sensitive or allergic patients.

**Note:** Please pass all designated information to the dentist, so that sensitivity or allergies can be consulted prior to use to treat a patient.

### 5. SIDE EFFECTS

With correct processing and application by the dentist and dental technician no side effects were observed. Immune reactions (for example, allergies) cannot be completely eliminated. Please report possible side effects to us.

### 6. MILLING

Use fine-toothed tungsten carbide burs to separate the milled restoration from the product. Cut the retaining bars between crown / bridge and disc carefully and without applying pressure. Restorations can be pre polished with suitable silicone polishes and goat hair brushes. To achieve a high shine polish, use the diamond polishing pastes recommend for the purpose. Thorough polishing is absolutely necessary to avoid plaque build-up.

## 7. CLEANING

After completion of the restoration, clean with compressed air.  
Cleaning with a steamer is not recommended. There is a possibility of deformation of the product due high pressure and heat. Clean with water in ultrasonic bath.

## 8. FIXING

Can be fixed with all temporary cements

## 9. PRODUCT DESCRIPTION

The product is used to fabricate temporary crowns and bridges to replace teeth or groups of teeth. Can be used in all open CAD/CAM systems. Available as a disc in various colours.

## 10. ADVANTAGES

- Very high material homogeneity thanks to the industrial polymerization process
- Well-balanced combination of fracture toughness and elasticity for clinical use. This guarantees excellent long-term stability
- Excellent abrasion resistance
- Ensures a high process safety, no mixing errors, no polymerization shrinkage and no unpleasant smell as no hand mixing or cartridge
- Economic fabrication due to digital workflow
- Lasting shades stability and aesthetics
- Natural fluorescence

## 11. TECHNICAL CUSTOMER SERVICE

### Manufacturer

on dent  2195  
Karacaoglan Mah. Erena Plaza 6166 Sk., No: 32/1J  
35070 Bornova / İZMİR / Turkey  
T: 0 232 253 25 77 / 0 546 459 10 09  
e-Mail: info@on-dent.com

### Global business partner

MINDFAB GmbH  
Max-Josef-Metzger-Str. 6  
86157 Augsburg, Germany  
T: +49 821 455252-0  
e-Mail: augsburg@mindfab.com

## 12. EXPLANATION OF SYMBOLS

Consult instructions for use

Batch Code

Catalogue number

Manufacturer

Keep away from sunlight / Keep in a dry place

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