Mindestwandstärke Exocad

LASERMELTING

FRÄSEN

3D-DRUCK

SERVICE

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The screenshot displays the Exocad software interface. On the left, there are two panels: 'Ein-/ausblenden' (Show/Hide) and 'Freiformen' (Freeforms). The 'Ein-/ausblenden' panel shows a list of layers with 'Mindestdicke' (Minimum thickness) highlighted in red. The 'Freiformen' panel shows various settings for freeform tools, including 'Höcker' (Bumps), 'Zahnbereiche' (Tooth areas), 'Ganzen Zahn' (Whole tooth), and 'Leisten' (Ridges). The main area shows a 3D model of a dental arch with a yellow highlight on a specific tooth. On the right, a vertical toolbar contains icons for 'Speichern' (Save), 'Experte' (Expert), 'Werkzeuge' (Tools), 'Abstände zeigen' (Show distances), and 'Schnittansicht' (Section view). An orange arrow points from the 'Mindestdicke' layer in the 'Ein-/ausblenden' panel to the 'Experte' icon in the toolbar. At the bottom right, there is a 'Neue Ansicht' (New view) button and a version number 'v3.0-7662/64'.

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The screenshot displays the Exocad software interface. On the left, a tree view shows the project structure with 'Mindestdicke' highlighted in red. An orange arrow points from this menu item to a context menu in the center. The context menu, titled 'Zahn 15 (Offset-Käppchen) - Käppchen', lists various functions. The option 'Mindestdicke festlegen...' is highlighted with a white background and a thin orange border. The background shows a 3D model of a dental crown preparation. On the right side of the interface, there is a vertical toolbar with icons for 'Speichern', 'Wizard', 'Werkzeuge', 'Abstände zeigen', and 'Schnittansicht'. At the bottom right, there is a 'Neue Ansicht' button and a navigation pad.

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The screenshot displays the Exocad software interface. On the left, a tree view under 'Ein-/ausblenden' shows 'Mindestdicke' selected. Below it, the 'Mindestdicke festlegen' dialog box is open, with 'Mindestdicke' set to 0,45 mm and 'OK' highlighted. An orange arrow points from the 'OK' button to the 3D dental model in the center. The model shows a dental arch with a green crown on the upper left tooth. On the right, a vertical toolbar contains icons for 'Speichern', 'Wizard', 'Werkzeuge', 'Abstände zeigen', and 'Schnittansicht'. At the bottom right, there is a 'Neue Ansicht' button and the version number 'v3.0-7662/64'.

CADdent® GmbH
Max-Josef-Metzger-Str. 6 | 86157 Augsburg

Telefon: +49 821 5999965-0
Fax: +49 821 5999965-44
E-Mail: augsburg@caddent.eu

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W W W . C A D D E N T . E U