dima[®] Print Gingiva Mask

GB Instructions for use dima[®] Print Gingiva Mask

Only for use by dental professionals.

1. Purpose
Photopolymer for the fabrication of gingiva masks for dental models.

- 1.1 Indications
Fabrication of gingiva masks for dental models.
- 1.2 Contraindications
Not for use in the mouth.
- 1.3 Potential side effects
This product or one of its components may in particular cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

2. Product features

- 2.1 Package sizes/Delivery forms
1000 g bottle
- 2.2 Supplies
For use with:
 - 3D printer, e.g. cara Print 4.0
 - Cleaning devices, e.g. cara Print Clean or ultrasonic bath
 - Light polymerization device, e.g. HiLite Power 3D
- 2.3 Composition
(Meth)acrylate-containing monomers, photo initiators, dyes

3. Product application

- a. Shake vigorously in the original container for five minutes before use.
Note: Inadequate shaking may result in color variations and printing errors.
- b. Print the objects according to the instruction manual supplied with the 3D printer.
- c. Pre-clean the printed objects for three minutes in Isopropanol using the appropriate supplies (see Item 2.2).
- d. Thoroughly remove remaining liquids using compressed air.
- e. Repeat step c using pure Isopropanol for two minutes.
Note:
 - The objects should not remain in the Isopropanol for more than five minutes, otherwise they will be damaged.
 - After cleaning, make sure the printed objects are dry and free from solvent residues.

- f. Place the printed object for final polymerization in the photo polymerization unit according to the table below.
Note: The final properties depend on the adherence to the recommended post-curing process.
We recommend using the Kulzer HiLite Power 3D light curing unit.

Materials	Pre-cleaning min	Post-cleaning min	Post-curing total time min
Gingiva Mask	3	2	10 (Rotate objects after 5 minutes)

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4. General information

Please carefully observe the following safety instructions as well as all processing instructions included in other sections.

This product is only to be processed in accordance with the instructions for use. Any alteration of these instructions during use is at the discretion of and under the sole responsibility of the dental professional.

- 4.1 Warnings and safety instructions
Suspected of damaging the unborn child. Toxic to aquatic life with long lasting effects. May cause an allergic skin reaction. May be irritating to eyes, respiratory organs and skin. Avoid contact with eyes. Avoid contact with skin. May cause sensitisation by skin contact. Do not inhale. Wear protective gloves/protective clothing/eye protection/face protection.
- 4.2 Storage conditions
Do not store above 25°C (77°F). Do not use the material after the expiration date. Avoid direct sunlight.
- 4.3 Disposal information
Recommendation: Dispose of in accordance with local regulations. Do not dispose of contents or partially emptied packaging in household waste or allow them to enter the sewage system. European Waste Catalogue: 180106 Chemicals consisting of or containing dangerous substances.

5. Product Questions

Please quote batch number and article number in all correspondence about the product.
Additional information can be found on the download page at: www.kulzer.com/cara-print

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© Kulzer 2020. Todos os direitos reservados. Este documento contém informações confidenciais e/ou protegidas por direitos autorais. É proibida a reprodução ou distribuição sem a autorização prévia por escrito da Kulzer. Este documento contém informações confidenciais e/ou protegidas por direitos autorais. É proibida a reprodução ou distribuição sem a autorização prévia por escrito da Kulzer.