Gebrauchsanweisung – Istruzioni d'uso Instruction manual – Instrucciones de uso Mode d'emploi

Deutsch

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(Lu. – Ven. ore 08:00 – 18:00)

Destined use:

TEMP BASIC is used for manufacturing of provisional crowns and bridges in the anterior and posterior tooth area.

TEMP PREMIUM is used for manufacturing of provisional crowns and bridge frameworks in the anterior and posterior tooth area. Due to its improved mechanical properties, it is especially suited for large-span bridges. TEMP PREMIUM FLEXIBLE and MULTISTRATUM® FLEXIBLE are used for manufacturing of bite splints and provisional crowns and bridge frameworks in the anterior and posterior tooth area. Due to the flexibility of the composite, manufacturing inaccuracies and possible slightly diverging insertion directions are being compensated. All materials can be processed with any Zirkonzahn Milling Unit using the appropriate tools and milling parameters. TEMP BASIC, TEMP PREMIUM, TEMP PREMIUM FLEXIBLE as well as MULTISTRATUM® FLEXIBLE are used for manufacturing of long-term provisionals. Once a year the attending dentist should examine the work and decide whether it can still remain in the mouth of the patient. The attending dentist is supposed to inform the patient in this regard before treatment. Due to the fact that after a certain period of time optical and mechanical defects/damages to the material can occur, Zirkonzahn recommends removing the work and replacing it with a new one.

Contraindications:

Do not use in conjunction with cloves, nutmeg, pimento and teas containing essential oils.

Working instructions:

- 1) Fix the resin disc in the clamping device provided for this purpose.
- 2) Check milled framework for obvious chips or cracks, use only flawless framework for temporary restorations.
- 3) Mechanical trimming of blanks is possible. The elaboration is carried out with tungsten-carbide burs. The polishing is carried out with diamond rubber wheels and pumice. The high gloss polishing is carried out with goat-hair brush and Composite Polishing Paste. Important: Do not overheat the resin when using burs, brushes, steam jets etc.
- 4) Before inserting the restoration, sandblast the fitting surface. Free the final job from dust (steam jets) and sterilize it.
- 5) Use temporary, eugenol-free cements for insertion.

Hints:

Resin discs are already polymerized when supplied, but avoid dropping them nevertheless. Product quality and durability can be compromised if handled improperly. Dust from grinding can irritate eyes, skin or mucous membranes, take protective measures.

Should intolerance to the product be suspected, use it only after an allergy screening test has been performed with issue of medical statement of non-allergy conditions. Disinfect the milled parts before inserting them in the patient's mouth. Implant hygiene is the patient's responsibility. This instruction for use is based on our current levels of knowledge. The patient will be held fully responsible as regards correct use of the product. The manufacturer cannot be held liable for any unsatisfactory results, given that they have no influence on processing procedures. The user (dental technician) shall inform the dentist with regards to the operations to be performed (such as sterilization for example) before inserting the part into the patient's mouth, making sure that the dentist has received all of the information contained in this instruction for use. Any claims for damages shall concern exclusively the commercial value of our products.

MULTISTRATUM® FLEXIBLE is a multilayer resin composed of a five-colour shading from dentine to enamel. The cervical side is the one with the adhesive label.

Recommended minimum dimensioning:

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Storage and durability:

Store in a dry place at room temperature, do not expose to physical impact or vibration, avoid contact with liquids, protect discs from contamination.

Handling:

Handling of this product only by trained personnel.

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