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LASERMELTING

FRESATO

STAMPA 3D

SERVIZI

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
The screenshot displays the Exocad software interface. In the center is a 3D model of a dental arch with a yellow crown on a tooth. On the left, there are two panels: 'Gruppi' (Groups) with 'Spessore min.' (Minimum thickness) selected, and 'Free-Forming' with various options like 'Cuspidi' (Cusps) and 'Parti del dente' (Parts of the tooth). On the right, a vertical toolbar contains icons for 'Salva' (Save), 'Esperto' (Expert), 'Strumenti' (Tools), 'Mostra distanza' (Show distance), and 'Vista in sezione' (Section view). An orange arrow points to the 'Esperto' icon. At the bottom right, there is a version number 'v3.0-7662/64'.

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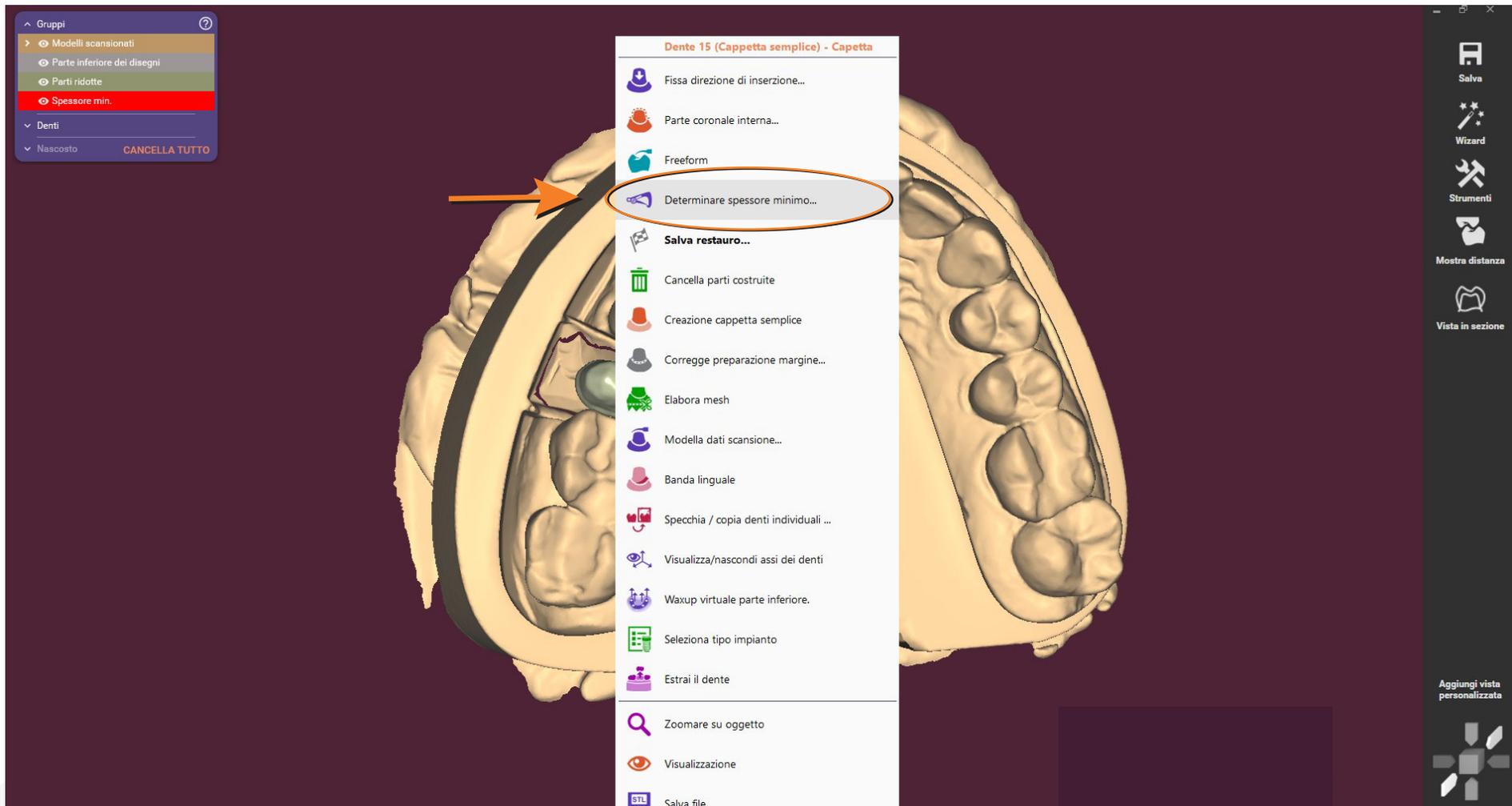The screenshot displays the Exocad software interface. In the center is a 3D model of a dental arch with a green crown on a tooth. On the left, a 'Gruppi' (Groups) panel shows a tree view with 'Spessore min.' selected. Below it, a dialog box titled 'Determinare spessore mi...' is open. The dialog has two sliders: 'Spessore min.' set to 0.45 mm and 'Alza in senso occlusale' set to 0 mm. The 'OK' button is highlighted with a red circle, and an orange arrow points from the 'Spessore min.' value to it. On the right side of the interface, there is a vertical toolbar with icons for 'Salva', 'Wizard', 'Strumenti', 'Mostra distanza', and 'Vista in sezione'. At the bottom right, there is a section for 'Aggiungi vista personalizzata' and a version number 'v3.0-7662/64'.

**CADdent® GmbH (sede operativa)**  
**Max-Josef-Metzger-Str. 6 | 86157 Augsburg | Germany**

**CADdent® S.r.l. (sede legale)**  
**Via Leonardo da Vinci 12 | 39100 Bolzano (BZ) | Italy**

Telefono: +39 0471 1660026  
E-mail: [italia@caddent.it](mailto:italia@caddent.it)

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WWW.CADDENT.IT